

Serenity Choice™

EN User Guide
BG Ръководство за потребителя
CS Návod k použití
DA Brugsanvisning
DE Gebrauchsanweisung
EL Οδηγός χρήσης
ES Instrucciones para el usuario
ET Kasutusjuhend
FI Käyttöohjeet
FR Mode d'emploi
HR Uput za uporabu
HU Felhasználói utasítás
IT Istruzioni per l'utente
LT Naudojimo instrukcija
LV Lietotāšanas pamācība
NL Gebruiksaanleiding
NO Bruksanvisning
PL Instrukcja obsługi
PT Instruções de utilização
RO Înstrucțiuni de utilizare
RU Руководство пользователя
SK Pokyny na používanie
SL Uporabniška navodila
SV Användningsinstruktioner

EN | English

USER INSTRUCTION

THE PRODUCT CONSISTS OF TWO PARTS: THE L SHAPED FILTER (ITEM G & H) AND THE REUSABLE EARPLUG (ITEM F). THE FILTER IS SUPLIFTED FITTED INTO THE LARGE EAR TIP (7-12mm). First insert the large ear tip into the ear canal by following the instructions (1. Insert into the ear tip). If the filter direction is incorrect, please turn it around and place the filter into the medium size ear tip (6-11 mm) following the instructions (3. Changing ear tip size).

Please note that it is not possible to achieve a proper seal with the small size filter (S-M-L). An extra-large size ear tip (8-9mm) is available on request.

1. Inserting Ear tip

1. ENSURE THE FILTER IS CORRECTLY POSITIONED IN THE EAR TIP (Image G & H).
 2. Grip the tongue of the ear tip between thumb and forefinger.
 3. Pull the tip of the ear outward and upward to ease insertion: This is best achieved by holding the ear with the opposite hand, the arm placed around the back of the head (Image D).
 4. Position the tongue at the top of the ear making it easier to find and grip when removing the tip from the ear (Image D).
 5. Gently push and twist the product into the ear until it sits comfortably in the ear canal without forming a good seal with the ear canal.

2. Removing, Cleaning and Storage

1. Find and grip the tongue of the ear tip (Image D).
 2. Remove the product with a slow twisting motion to gradually break the seal with the ear canal. This will eliminate any discomfort that may occur during removal.
 3. Clean the ear tip after each use to remove earwax (Cerumen) and other debris: the ear tips can be wiped clean with a damp cloth or antibacterial cleaning tissues.
 4. Always store the ear tips in a dry, clean and airtight container (e.g. a bag). Do not store the ear tips in plastic products or containers.
 Recommended storage temperature is range between -10 °C and 50 °C.

After each use of the filter (Changing ear tip size), the ear tips can be cleaned with soap or hygiene tablets and then rinsed in tepid water and dried thoroughly.
 We recommend only cleaning the filter if necessary. Do not use alcohol-based cleaning solutions, soaps or detergents. After cleaning, allow the ear tips to thoroughly rinse in tepid water and allowed to dry slowly.

3. Changing ear tip size

1. Remove the filter from the current ear tip: Gently squeeze the ear tip just below the filter until it 'pops' out of the ear tip (Image A).
 2. Before placing the filter ensure that the logo in the filter is facing you so that it will face out of the ear tip once inserted (Image A & B).
 3. Insert the filter into the larger opening of the ear tip: This is best achieved by inserting the filter at an angle, pushing the filter into the ear tip (first Image F).

4. Press down on the logo to correctly position the filter in the cavity of the ear tip (Image G, H & I).
 5. Insert the ear tip into the ear canal by following the instructions (1. Inserting into the ear).

WARNINGS
 If these instructions and warnings are not adhered to, the product may be affected by the product will be severely impaired.

The product is intended to be placed in the ear and to protect the wearer from hazardous noise levels. The product is not intended for any other use.

CHILDREN'S HAZARD - Keep the product out of reach of young children.

The product must be fitted, adjusted and maintained in accordance with the instructions. Improper fitting can reduce its effectiveness.

In order to achieve the designed attenuation level, it's important that the outside of the ear tip has an airtight seal with the ear canal.

If the product is not sealed when coming into contact with water, ears or alcohol.

Please ensure that the product and both hands are clean before insertion in order to reduce the risk of infection.

Sharing the product can increase the risk of infection.

Do not share the product with other people.

One reason for evaluating whether the use of the product affects the ability to safely perform an activity, Stop the activity when the product is disruptive or disruptive.

It is not recommended to use the product after the expiration date (see outside of package). The specified attenuation of the product can no longer be guaranteed after the expiration date.

LIMITED WARRANTY
 Phonak Communications AG offers a 2 (two) years limited international warranty, valid as of the date of purchase. This warranty is valid only if proof of purchase is shown.

Warranty limitation: This warranty does not cover damage from improper handling, or care exposure to chemicals or undue stress. Damage caused by third parties or non-authorized service centers renders the warranty null and void.

CS | Česká

Phonak Serenity Choice™ Reusable Earplugs

APPLICATIONS



VERSION: PR-205SP / 07.2020



